

Airtraq

Universal Adapter for Smartphones

INSTRUCTIONS FOR USE

ENGLISH

PRODUCT CODE AND DESCRIPTION

Code: A-307

Description: Airtraq Universal Adapter for Smartphones

BASIC INFORMATION

The A-307 has been specially designed to work only as an accessory for the Airtraq Video Laryngoscope.

It fits most Smartphones even using a protective case or a protective sleeve.

A-307 has always to be used with "Airtraq Mobile" App installed in the Smartphone.

"Airtraq Mobile" App provides colour and real-time images on the Smartphone's screen and allows to record video and take snapshots keeping them into a library and allowing the user to edit information related to each video or snapshot.

"Airtraq Mobile" App is available for iOS (iPhones) and Android devices. It can be downloaded free of charge from App StoreSM or Google PlayTM.


Airtraq Universal Phone Adapter needs to be initially adjusted for each specific Phone model with or without a protective case. For regular use, once the adapter has been set up, inserting the Phone into the adapter is quick and simple.

INSTALLING AIRTRAQ MOBILE APP INTO SMARTPHONE


Before starting to use a Smartphone with A-307, download from App StoreSM or Google PlayTM "Airtraq Mobile" app and install it.

"Airtraq Mobile" requires iOS v7.0 and up or Android 4.0 and up.

HOW THE ADAPTER WORKS

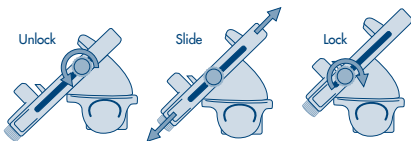


Unlock knob / Slide lower tab

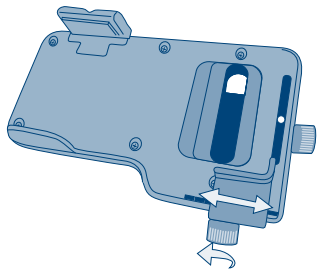


INITIAL SET-UP


- Insert phone with camera to the right

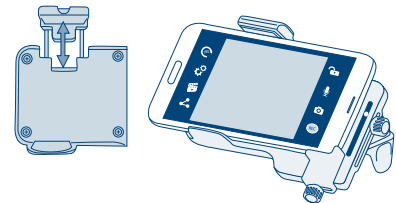


- Slide phone to center camera in slot

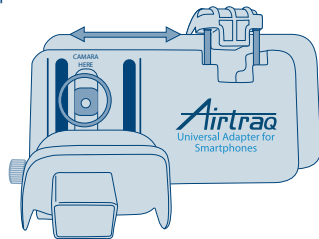


- Slide lower tab
- Block knob

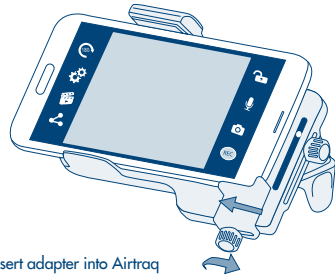


- Insert adapter into Airtraq
- Turn on Airtraq

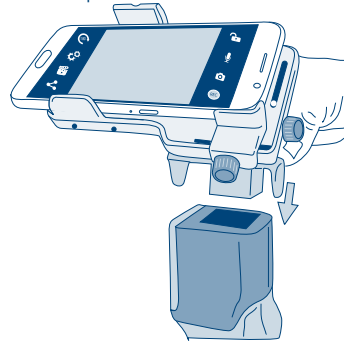

- Go to Settings > Phone Alignment Aid
- Hold Airtraq and slide Adapter until white line is aligned with white dot
- Block knob

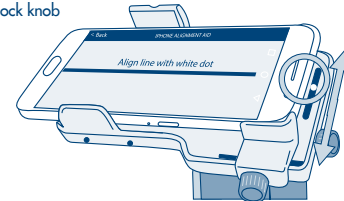


- Go to main screen (live video)
- Slightly unlock knobs
- Fine tune alignment
- Lock knobs

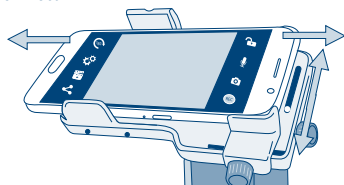

AIRTRAQ MOBILE FUNCTIONS

Image control

Zoom Drag Focus

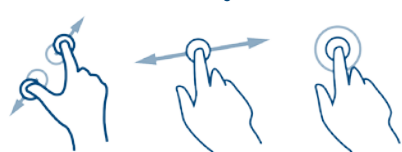

Menus

Main menu functions:

- : Image can be rotated 180 degrees when necessary using the Rotate icon.
- : Shows settings menu functions.
 - Phone alignment aid: Helps the user to align the Smartphone.
 - Quick Start Guide: Shows how the Adapter works.
 - Demo Video: Shows a demonstration video using the Adapter during an intubation.
 - Airtraq Web: Offers a direct access to Airtraq website.
 - Requested Info / Suggestions: Allows user to communicate with Prodol.
 - About: Shows basic information about the App.
- : Shows a list of the recorded videos and snapshots. Allows the user to edit the information related to each snapshot or video.
- or : Allows user to share the App.
- : Locks zoom and image location.
- : Enables/disables audio recording.
- : Snapshots.
- : Video recording.

MOUNTING A-307 ONTO THE AIRTRAQ

To prepare A-307 for intubation.

- Remove eyecup from Airtraq.
 - Insert A-307 onto Airtraq.
 - Insert Smartphone into A-307. Orientate the Smartphone so that its camera is on top of the Airtraq.
- To remove A-307 from the Airtraq once intubation is finished:
- Pull A-307 away from Airtraq.
 - Remove Smartphone from A-307.

APP PERMISSIONS (iOS ONLY)

Airtraq Mobile app will ask for some permissions the first time is launched. The app will lose functionality if the user does not agree with the permissions.

To manage these permissions, open the Settings app and scroll down to the list of apps at the bottom. Tap on Airtraq mobile to see the permissions.

CLEANING INSTRUCTIONS

The A-307 is a non-critical device. Recommended cleaning is low-level disinfection between each patient use. Be sure to follow your institution's specific cleaning procedures in consultation with this manual.

1. Remove A-307 from the Airtraq.
 2. Remove Smartphone from A-307.
 3. Cleaning: Use clean cotton gauze pads that are saturated with the cleaning solution to wipe down the exterior surfaces of the A-307. Use soft brushes with the cleaning solution to remove any residues from areas that cannot be reached with the gauze pads. Be sure to remove all visible contaminant or foreign matter. Be careful to keep running liquid off the surfaces. The following cleaning solutions may be used:
 - a. Enzymatic Cleaning Solutions (e.g.: ENZOLTM Enzymatic Detergent).
 - b. Neutral pH soap and water.
 - c. Sodium bicarbonate solution (8- 10 %).
 4. The following disinfection agents may be used:
 - a. Peracetic Acid Solution (0.08%).
 - b. Isopropyl Alcohol (70%).
 - c. Solution containing 70% isopropyl alcohol and 2% chlorhexidine (e.g. Clinell Wipe).
 - d. Solution containing chlorine dioxide (e.g. Tristel Wipe).
 - e. PDI Sani-Cloth[®] Germicidal Wipes (AF3, Bleach, Plus or Super Sani-Cloth[®]).
 5. Blot dry the A-307 using an individual sterile surgical towel.
- Caution:
- Avoid touching the mirror inside A-307.
 - Do not autoclave.
 - Do not rinse under running water.
 - Do not soak in liquids.
 - Avoid liquid or moisture from going into the A-307.

PATIENT PRIVACY

It is the responsibility of the physician/user to ensure the security of medical information generated with the use of a cell phone camera.

MANUFACTURER'S WARRANTY

Warranty & Service

The manufacturer warrants this device against faulty materials or manufacturing defects for a period of one (1) year from the date of purchase, provided that it is used in accordance with the procedures set up in the instructions for

use. This warranty is only applicable if the device has been bought from an authorized distributor.

The manufacturer disclaims all other warranties, whether expressed or implied, including, without limitation the warranties of merchantability or fitness for a particular purpose.

The warranty covers free repair or swap for an equivalent device of a defective device which fault is proven to be consequence of the manufacturing process.

If you have any technical problems with the device, please call the Airtraq Distributor in your country or send a message through our web site (www.airtraq.com). Before making your call please have the lot number ready. The lot number of the A-307 is located on a sticker in the lower side of original package.

In case of a hardware defect confirming the warranty conditions, your device will be collected and you will receive another one within 20 working days after we receive your detected one.

Exclusions from Warranty

Damage, deterioration or malfunction resulting from:

1. Accident.
2. Misuse, neglect, fire, water, lightning or other acts of nature.
3. Repair or attempted repair by anyone not authorized by Supplier.
4. Modification by anyone not authorized by Supplier.
5. Any damage of the product due to shipment.
6. Causes external to the product, such as; over powering the unit, electric power fluctuations or failure, pouring liquids over the device, etc.
7. Normal wear and tear.
8. Any other causes which does not relate to a product defect.

Handling Instructions for Defective Devices

1. Return units must be packed in original and secured packaging materials. Damaged incurred during transportation from the customer site is solely the responsibility of the sender.
2. Customer must provide a contact name, phone number, shipping address, lot number of the A-307, description of the problem and proof of purchase.
3. In all cases when the device shows defects which are not covered by the warranty, the customer will remain responsible for all handling costs and transportation fees.

Manufactured by:

PRODOL MEDITEC LIMITED

No. 18, 7th Science Ave.,
Hi-Tec Coast, Zhuhai, Guangdong
519085 P.R. China

EU Representative:

PRODOL MEDITEC, S.A.

Muelle Tomás Olavarrí 5, 3^º
48930 Las Arenas (Vizcaya)
SPAIN

For further advice on using the A-307 visit:
www.airtraq.com or contact:

America: +877-624-7929
info.usa@airtraq.com
EU & Other: +34944804690
info@airtraq.com

Airtraq is a registered trademark.

Apple and the Apple logo, iPad and iPhone are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc.

Android and Google Play are trademarks of Google Inc. All other trademarks and copyrights are the property of their respective owners.

