Airtraq Avant

VIDEO LARYNGOSCOPE

BLADE & EYECUP

INSTRUCTIONS FOR USE

ENGLISH


DESCRIPTION

The Airtraq Avant is a video laryngoscope designed to facilitate intubation. It allows full visualization of the airway, during 100% of the intubation. It does not require hyperextension of the neck and permits intubating patients in virtually any position.


COMPONENTS

Airtraq Avant comprises 3 elements that have to be assembled by the user every time an intubation is to be performed:

The Optics: A **REUSABLE** piece that contains the optical, the anti-fog and the electronic systems and that is articulated to facilitate its insertion into the blade; The Optics element only works when it is inserted into a Blade.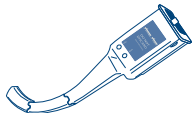

The Blade: A **DISPOSABLE** rigid piece of plastic, anatomically shaped, that consists of two side by side channels. One channel, which terminates in a distal lens, for insertion of the optics, and the other channel, open at its distal end, acts as a guide for the endotracheal tube.

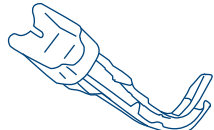The Eyecup: A **DISPOSABLE** piece that is assembled on top of the blade and which also has a proximal lens.

Airtraq Avant Blade and Eyecup are packed together in an individual pack and are delivered ready to be used.

BLADE SIZES


Regular: A-511. Size 3
For use with ETT 7.0 – 8.5. Minimum patient mouth opening: 18 mm
Small: A-521. Size 2
For use with ETT 6.0 – 7.5. Minimum patient mouth opening: 18 mm

USING THE AIRTRAQ AVANT BLADE


PREPARATION

- Select the appropriate blade size based on the size ETT to be used and remove the eyecup.

- Place the eyecup over the proximal end of the Optics.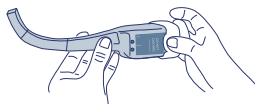


- Insert the Optics into the blade fully, until it clicks into position.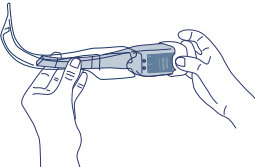

- Push the On/Off switch located on the side of the eyecup. The light will start blinking for approximately 60 seconds until the anti-fog system warms the lens of the blade. Once the device is ready for intubation the light will become steady.

AIRTRAQ AVANT OPTICS REMOVAL FROM BLADE AND DISPOSAL

- Separate the Airtraq Avant Optics from the Blade by firmly gripping both lateral sides of the eyecup and pulling apart. Make sure the Optics do not become in contact with any potentially contaminated surface.

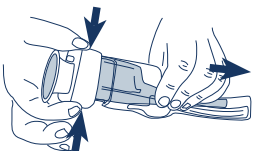- Discard the disposable blade and eyecup as any other potentially contaminated waste.
- If needed, place the Optics back onto the Docking Station to recharge the battery.

USAGE TIPS

- Initial experience should be gained in non-difficult airways.
- Insert the Airtraq Avant, avoiding the tongue, and slide it softly and slowly.
- Keep the Airtraq Avant in the mouth's midline.
- Look through the eyecup before the Airtraq Avant gets to the vertical plane.
- If structures (arytenoids, epiglottis, etc.) are not clearly recognized, withdraw the Airtraq Avant slightly.
- Once the tip is located at the epiglottis, either at the vallecula (Macintosh style), or under the

epiglottis (Miller style), gently lift up the Airtraq Avant (do not tilt or use a lever action).
7. Advance the ETT slowly.

WARNINGS AND PRECAUTIONS

- This product should only be used by personnel trained in insertion of endotracheal tubes.
- Do not put pressure on the teeth with this device.
- Do not force the Airtraq Avant into the upper airway.
- Use only with non-flammable anesthetics.

AVANT BLADE AND EYECUP DISPOSAL

After being used, discard the Airtraq Avant Blade and Eyecup as any other potentially contaminated waste.

MANUFACTURER'S WARRANTY

The manufacturer warrants the Airtraq Avant Blade and Eyecup against faulty materials or manufacturing defects for only one use or until the expiration date, whichever comes first, provided that the Airtraq Avant is used in accordance with the procedures set forth in these instructions. This Warranty is applicable only if the device is purchased from an authorized distributor.

The AIRTRAQ AVANT BLADE and EYECUP are designed for SINGLE PATIENT USE.

Warning! Cleaning and reuse of the AIRTRAQ AVANT BLADE may compromise patient safety.

Use of Airtraq Avant Blades that have been cleaned or sterilized after previous use may generate serious consequences in the product's performance and will void the Airtraq Avant warranty. The manufacturer disclaims all other warranties, whether expressed or implied, including, without limitation, the warranties of merchantability or fitness for a particular use.

Manufactured by:
PRODOL MEDITEC LIMITED
No. 18, 7th Science Ave.
Hi-Tec Coast, Zhuhai, Guangdong
519085 P.R. China

USA Representative: **AIRTRAQ LLC**
800 Sun Park Dr.
Fenton, MO 63026 USA

EU Representative:
PRODOL MEDITEC, S.A.
Muelle Tomás Olavarri 5, 3º
48930 Las Arenas SPAIN

Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.

For further advice on using the Airtraq Avant please visit:
www.airtraq.com or contact:
info@airtraq.com or
info.usa@airtraq.com

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AIRTRAQ is a registered trademark.