



VIDEO LARYNGOSCOPE FOR NASO-TRACHEAL INTUBATION

US Patent No 6,843,769

INSTRUCTIONS FOR USE

REF: A061; A861

INTENDED USE AND DESCRIPTION

Airtraq SP is a video laryngoscope to facilitate insertion of an endotracheal tube (ETT) during nasal intubation. It allows visualization of the airway during intubation. It minimizes hyper extension of the neck and permits intubating from virtually any position.

It is a **SINGLE USE** medical device provided clean and ready to use.

To perform its purpose Airtraq SP must have attached to its proximal end, either

- its eyecup or
- the WiFi Camera (Ref A-390) offered by the manufacturer, attached after removing its eyecup, or
- an Endoscopic camera attached to the eyecup.

SIZES AND SPECIFICATIONS

Naso-Tracheal: A-061

Orange.

Minimum patient mouth opening: 15 mm

For use with any standard ETT used in naso-tracheal intubations.

Maximum insertion portion width: 27.8 mm.

Working Length: 117.3 mm.

Field of View: Vertical 32 degrees; Horizontal 27 degrees

Direction of View: Vertical 12 degrees; Horizontal 22 degrees

USE INSIDE OF THE MRI ENVIRONMENT

This medical device was determined to be MR conditional according to the terminology specified by ASTM Standard Practice for Marking Medical Devices and Other Items for Safety in the Magnetic Resonance Environment.

The device can be used in the MRI environment according to the following conditions:

- Static magnetic field of 3-Tesla or less
- Spatial gradient magnetic field of 720-Gauss/cm or less

IMPORTANT NOTE: The device is intended for use inside of the MRI environment (e.g., in the MR system room). It will not be

utilized directly inside of the MR system (e.g., inside of the bore of the scanner), during its operation (i.e., scanning). As such, the assessment of magnetic field interactions for the device specifically involved evaluations of translational attraction in relation to exposure to a 3-Tesla MR system, only.

TECHNIQUE FOR USING THE AIRTRAQ SP NT


I. PREPARATION & TEST

- Press orange switch located below the battery cover to Turn On the light.
- After 30 seconds check that the light is steady. If so, anti-fog system is fully activated.

NOTE: If light automatically turns off, the unit has become defective and must not be used.

II. AIRTRAQ SP NT PLACEMENT (Fig. 1)

- Insert the Airtraq SP NT into the midline of the patient's mouth. Take special care to avoid pushing the tongue inside the oropharynx.
- Before it reaches the vertical plane, begin looking to identify airway structures.
- Continue insertion until the epiglottis is identified. Place the tip of the Airtraq SP NT in the vallecula. Alternatively, the tip can be placed under the epiglottis, lifting it out of the way.
- Gently lift up the Airtraq SP NT to expose the vocal cords.

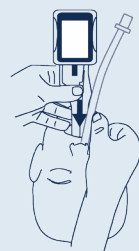


Slide midline

- ✓ Do not insert too deep

III. ETT INSERTION (Fig. 2)

- **Align the vocal cords in the center of the visual field** by gently moving the tip of the Airtraq SP NT as needed.
- Insert the ETT through a nostril as in any standard naso-tracheal intubation, and gently advance it through the nostril. If needed rotate ETT. Check insertion depth.
- Inflate the ETT cuff as normal and check for proper positioning. In some situations, other airway tools may be needed to guide the ETT through the vocal cords.

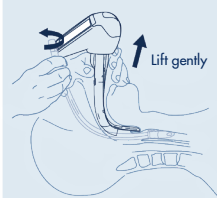


Center glottis
Advance ETT

- ✓ Lift gently
- ✓ Twist Airtraq to center vocal cords
- ✓ Corkscrew ETT

IV. AIRTRAQ SP NT REMOVAL (Fig. 3)

- Hold the ETT in position and rotate back the Airtraq SP NT to remove it.

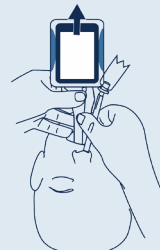

Remove midline

WARNINGS AND PRECAUTIONS

- This product should only be used by personnel trained in naso-tracheal intubations.
- Do not sterilize.
- Do not incinerate unless batteries have been removed.
- Do not submerge in liquids.
- Use only with non-flammable anesthetics.
- Do not put pressure on the teeth with this device.
- Do not force the Airtraq SP NT into the upper airway.

USAGE TIPS

1. Initial experience should be gained in non-difficult airways as well as with the standard Airtraq SP.
2. Insert the Airtraq SP NT, avoiding the tongue, and slide it softly and slowly.

- 3. Keep the Airtraq SP NT in the mouth's midline.
- 4. Look before the Airtraq SP NT reaches the vertical plane.
- 5. Do not insert too deep. If structures (arytenoids, epiglottis, etc.) are not clearly recognized, or ETT cannot be inserted, withdraw the Airtraq SP NT slightly.
- 6. Once the tip is located at the epiglottis, either at the vallecula (Macintosh style), or under the epiglottis (Miller style), gently lift up the Airtraq SP NT (do not tilt or use a lever action).
- 7. Advance the ETT slowly. If needed rotate ETT.

BATTERY CHARACTERISTICS

Each Airtraq SP NT is equipped with two AAA batteries in serial connection that provide a voltage of 3 volts. The batteries provide power to the LED light and to the anti-fog system when the switch is turned on. They should not be replaced.

STORAGE, TRANSPORT, SHELF LIFE AND SERVICE LIFE

The Airtraq SP NT should not be used, stored or transported at temperatures below -5°C/23°F or over 55°C/131°F. The relative humidity must not exceed 95%. The air pressure must not exceed 500 to 1060 hPa.

Airtraq SP NT shelf life is limited to the expiration date. Airtraq SP NT service life is limited to 40 accumulative minutes, 5 minutes before its end; the Airtraq SP NT will start blinking for 2 seconds every 20 seconds. Once maximum operating time is reached the Airtraq SP NT will blink continuously until battery is depleted.

DISPOSAL

- To dispose the Airtraq SP NT once it has been used:
- Separate the eyecup from the main body by pulling it up.
 - Remove the battery cover by pulling it away from the main body (pull away from the small notches).
 - Remove the batteries from the Airtraq SP NT and place them in an appropriate battery recycling container (dispose of them according to established recycling policies). The batteries are classified as non hazardous waste material and comply with European Directive WEEE. However, the manufacturer recommends separating them from standard trash.
 - Discard the Airtraq SP NT as any other potentially contaminated waste.

MANUFACTURER’S WARRANTY

The manufacturer warrants the Airtraq SP NT against faulty materials or manufacturing defects for only one use or until the expiration date, whichever comes first, provided that the Airtraq SP NT is used in accordance with the procedures set forth in these instructions. This Warranty is applicable only if the device is purchased from an authorized distributor.


This device has not been designed to be cleaned or sterilized. Use beyond this recommendation may generate serious consequences in the product's performance and will void the Airtraq SP's warranty. The manufacturer disclaims all other warranties, whether expressed or implied, including, without limitation, the warranties of merchantability or fitness for a particular use.

USER ASSISTANCE INFORMATION

Instructions for use are available online at <https://www.airtraq.com/IFU>
Visit www.airtraq.com for further advice on using this device
Any serious incident that occurs in relation to this device should be reported to the manufacturer and the competent Health Authority in which the user and/or patient is established. For

communication with manufacturer, please email user.assistance@airtraq.com, or:

- 1. Contact directly with Manufacturer at its address or by phone:
USA & Canada: +1877-624-7929
EU & Other: +34944804690
- 2. Contact Manufacturer’s Representative for your area (details below) or
- 3. Contact your Local Distributor

 **PRODOL MEDITEC LIMITED**
1/F, 4/F, Block C,
No. 18, 7th Science Ave.,
Zhuhai, Guangdong,
519085 P.R. China

 **& EUROPE**

PRODOL MEDITEC S.A.
Muelle Tomás Olavarri 5, 3º
48930 Las Arenas. SPAIN

UK Responsible Person:
Avena
Pure Offices, Plato Close,
Tachbrook Park,
Warwick, CV34 6WE, UK

USA & Canada Representative:
D. A. Daniel
2414 Lawton Ln.
Rowlet, TX 75089, USA

Airtraq is a registered trademark.
































Rx Only


www.airtraq.com/ifu



GLOSSARY OF SYMBOLS

SYMBOL	SYMBOL MEANING	SYMBOL	SYMBOL MEANING	SYMBOL	SYMBOL MEANING
	Reference Number		Number of Units		Legal Manufacturer
	Batch code		Manufacturing Date		Use-By date
	Authorized representative in the European Community		Importer		Medical device
	Fragile, handle with care		Consult instructions for use		Do not re-use
	Temperature limit.		Humidity limitation		Atmospheric Pressure Limitation
	Not Made with Natural Rubber Latex		Keep Dry / Protect from moisture		Federal (USA) law restricts the use of this device to sale by or on the order of a physician
	CE Mark, European Conformity		EAC Mark, Eurasian Conformity		UKCA Mark, UK Conformity Assessment
	This side up		Do not use if package is damaged		Distributor
	Separate collection for waste of electrical and electronic equipment		Type BF applied part		Australian sponsor
	Magnetic resonance conditional		Box/packaging recyclable		