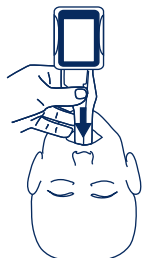
Airtraq^{sp}

VIDEO LARYNGOSCOPE FOR NASO-TRACHEAL INTUBATION

US Patent No 6,843,769

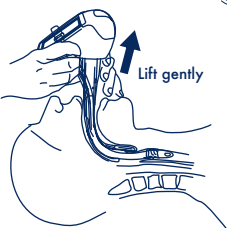
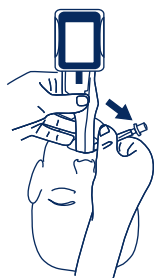
INSTRUCTIONS FOR USE

ENGLISH Naso-tracheal Intubation

Slide midline

- Do not insert too deep
- Lift gently
- Twist Airtraq to center vocal cords
- Corkscrew ETT

Center glottis
Advance ETT

Remove midline

DESCRIPTION AND INDICATIONS

The Airtraq SP NT is a SINGLE USE video laryngoscope designed to facilitate insertion of an endotracheal tube (ETT) during nasal intubation. It allows full visualization of the airway, during 100% of the intubation. It does not require hyper extension of the neck and permits intubating patients in virtually any position. It is provided clean, ready to use. Visualization can be performed directly through the eyecup or connecting it to any Endo Cam or to the accessories offered by the manufacturer.

SIZES

Naso-Tracheal: A-061

Orange. Minimum patient mouth opening: 15 mm

For use with any standard ETT used in naso-tracheal intubations.

TECHNIQUE FOR USING THE AIRTRAQ SP NT

I. PREPARATION & TEST

- Press orange switch located below the battery cover to Turn On the light.
- After 30 seconds check that the light is steady. If so, anti-fog system is fully activated.

NOTE: If light automatically turns off, the unit has become defective and must not be used.

II. AIRTRAQ SP NT PLACEMENT (Fig. 1)

- Insert the Airtraq SP NT into the midline of the patient's mouth. Take special care to avoid pushing the tongue inside the oropharynx.
- Before it reaches the vertical plane, begin looking to identify airway structures.
- Continue insertion until the epiglottis is identified. Place the tip of the Airtraq SP NT in the vallecula. Alternatively, the tip can be placed under the epiglottis, lifting it out of the way.
- Gently lift up the Airtraq SP NT to expose the vocal cords.

III. ETT INSERTION (Fig. 2)

- Align the vocal cords in the center of the visual field** by gently moving the tip of the Airtraq SP NT as needed.
- Insert the ETT through a nostril as in any standard naso-tracheal intubation, and gently advance it through the nostril. If needed rotate ETT. Check insertion depth.
- Inflate the ETT cuff as normal and check for proper positioning.
- In some situations, other airway tools may be needed to guide the ETT through the vocal cords.

IV. AIRTRAQ SP NT REMOVAL (Fig. 3)

- Hold the ETT in position and rotate back the Airtraq SP NT to remove it.

WARNINGS AND PRECAUTIONS

- This product should only be used by personnel trained in naso-tracheal intubations.
- Do not sterilize.
- Do not incinerate unless batteries have been removed.
- Do not submerge in liquids.
- Use only with non-flammable anesthetics.
- Do not put pressure on the teeth with this device.
- Do not force the Airtraq SP NT into the upper airway.

USAGE TIPS

- Initial experience should be gained in non-difficult airways as well as with the standard Airtraq SP.
- Insert the Airtraq SP NT, avoiding the tongue, and slide it softly and slowly.
- Keep the Airtraq SP NT in the mouth's midline.

4. Look before the Airtraq SP NT reaches the vertical plane.

5. Do not insert too deep. If structures (arytenoids, epiglottis, etc.) are not clearly recognized, or ETT cannot be inserted, withdraw the Airtraq SP NT slightly.

6. Once the tip is located at the epiglottis, either at the vallecula (Mcintosh style), or under the epiglottis (Miller style), gently lift up the Airtraq SP NT (do not tilt or use a lever action).

7. Advance the ETT slowly. If needed rotate ETT.

BATTERY CHARACTERISTICS

Each Airtraq SP NT is equipped with two AAA batteries in serial connection that provide a voltage of 3 volts. The batteries provide power to the LED light and to the anti-fog system when the switch is turned on. They should not be replaced.

STORAGE, TRANSPORT, SHELF LIFE AND SERVICE LIFE

The Airtraq SP NT should not be used, stored or transported at temperatures below -5°C/23°F or over 55°C/131°F. The relative humidity must not exceed 95%. The air pressure must not exceed 500 to 1060 hPa.

Airtraq SP NT shelf life is limited to the expiration date.

Airtraq SP NT service life is limited to 40 accumulative minutes, 5 minutes before its end; the Airtraq SP NT will start blinking for 2 seconds every 20 seconds. Once maximum operating time is reached the Airtraq SP NT will blink continuously until battery is depleted.

DISPOSAL

To dispose the Airtraq SP NT once it has been used:

- Separate the eyecup from the main body by pulling it up.
- Remove the battery cover by pulling it away from the main body (pull away from the small notches).
- Remove the batteries from the Airtraq SP NT and place them in an appropriate battery recycling container (dispose of them according to established recycling policies). The batteries are classified as non hazardous waste material and comply with European Directive WEEE. However, the manufacturer recommends separating them from standard trash.
- Discard the Airtraq SP NT as any other potentially contaminated waste.

MANUFACTURER'S WARRANTY

The manufacturer warrants the Airtraq SP NT against faulty materials or manufacturing defects for only one use or until the expiration date, whichever comes first, provided that the Airtraq SP NT is used in accordance with the procedures set forth in these instructions. This Warranty is applicable only if the device is purchased from an authorized distributor.

The Airtraq SP NT is designed for SINGLE-PATIENT USE.

This device has not been designed to be cleaned or sterilized. Use beyond this recommendation may generate serious consequences in the product's performance and will void the Airtraq SP's warranty. The manufacturer disclaims all other warranties, whether expressed or implied, including, without limitation, the warranties of merchantability or fitness for a particular use.

Manufactured by:

PRODOL MEDITEC LIMITED

1/F, 4/F, Block C
No. 18, 7th Science Ave.,
Zhuhai, Guangdong. 519085 China
USA Representative: AIRTRAQ LLC
2414 Lawton Ln.
Rowlett, TX, 75089, USA

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Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.

For further advice on using the Airtraq SP please visit: www.airtraq.com or contact: info@airtraq.com or info.usa@airtraq.com

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Airtraq is a registered trademark.
Representante en EEUU: AIRTRAQ LLC
PRODOL MEDITEC, S.A.