Airtraq^{sp}

VIDEO LARYNGOSCOPE FOR NASO-TRACHEAL INTUBATION

US Patent No 6,843,769
INSTRUCTIONS FOR USE

ENGLISH Naso-tracheal Intubation

Fig 1

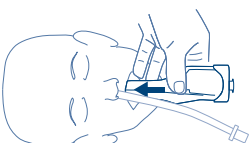
Fig 2

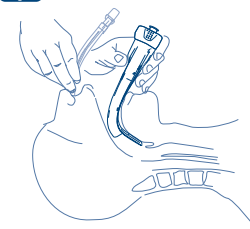
Fig 3

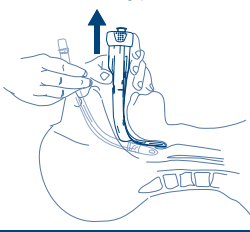
Fig 4


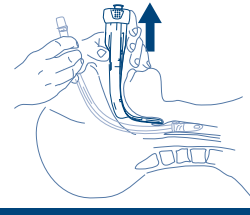
Fig 5

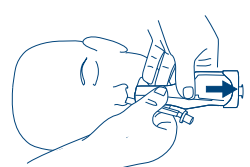DESCRIPTION AND INDICATIONS

The Airtraq SP NT is a SINGLE USE video laryngoscope designed to facilitate insertion of an endotracheal tube (ETT) during nasal intubation. It allows full visualization of the airway, during 100% of the intubation. It does not require hyperextension of the neck and permits intubating patients in virtually any position. It is provided clean, ready to use.

SIZES

Naso-Tracheal: A-061/ATQ-061

Orange. Minimum patient mouth opening: 15 mm

For use with any standard ETT used in naso-tracheal intubations.

TECHNIQUE FOR USING THE AIRTRAQ SP NT

I. PREPARATION

Turn On the light. Switch located below the battery cover. If no light is seen, replace unit.

NOTE: The light stops blinking (30-60 sec) when the anti-fog system is fully activated.

Prepare the ETT as usually for naso-tracheal intubations.

Appropriately lubricate the distal end of the Airtraq SP NT without contacting the lens.

II. AIRTRAQ SP NT PLACEMENT

• Insert the Airtraq SP NT into the midline of the patient's mouth. Take special care to avoid pushing the tongue inward (Fig 1).

• In some circumstances it may help to introduce it using the same technique as with an oral airway. In this case, always rotate the Airtraq SP NT clockwise from the clinician's perspective.

• Slide the Airtraq SP NT through the oropharynx keeping it in the midline.

• Before it reaches the vertical plane, begin looking through the eyecup to identify airway structures (Fig 2).

• Continue insertion until the epiglottis is identified. Place the tip of the Airtraq SP NT in the vallecula. Alternatively, the tip can be placed under the epiglottis, lifting it out of the way.

• Gently lift up the Airtraq SP NT to expose the vocal cords (Fig 3).

III. ETT INSERTION

• **Align the vocal cords in the center of the visual field** by gently moving the tip of the Airtraq SP NT as needed.

• Insert the ETT through a nostril as in any standard naso-tracheal intubation, and gently advance it through the nostril until it is visualized passing through the vocal cords. Check insertion depth (Fig 4).

• Inflate the ETT cuff as normal and check for proper positioning. Reposition, if needed.

• In some situations, other airway tools may be needed to guide the ETT through the vocal cords.

IV. AIRTRAQ SP NT REMOVAL

• Hold the ETT in position and rotate back the Airtraq SP NT to remove it (Fig 5).

WARNINGS AND PRECAUTIONS

• If utilizing any technique that requires rotation of the Airtraq SP NT, always rotate clockwise from the clinician's perspective.

• This product should only be used by personnel trained in naso-tracheal intubations.

• Do not sterilize.

• Do not incinerate unless batteries have been removed.

• Do not submerge in liquids.

• Use only with non-flammable anesthetics.

• Do not put pressure on the teeth with this device.

• Do not force the Airtraq SP NT into the upper airway.

USAGE TIPS

1. Initial experience should be gained in non-difficult airways as well as with the standard Airtraq SP.

2. Insert the Airtraq SP NT along the tongue, without pushing the tongue inward.

3. Keep the Airtraq SP NT in the mouth's midline.

4. Look through the eyecup before the Airtraq SP NT reaches the vertical plane.

5. If structures (arytenoids, epiglottis, etc.) are not clearly recognized, or ETT cannot be inserted, withdraw the Airtraq SP NT slightly.

6. Once the tip is located at the epiglottis, either at the vallecula (Macintosh style), or under the epiglottis (Miller style), gently lift up the Airtraq SP NT (do not tilt or use a lever action).

7. Advance the ETT slowly and twist it if needed.

BATTERY CHARACTERISTICS

Each Airtraq SP NT is equipped with two AAA batteries in serial connection that provide a voltage of 3 volts. The batteries provide power to the LED light and to the anti-fog system when the switch is turned on. They should not be replaced.

STORAGE, TRANSPORT, SHELF LIFE AND SERVICE LIFE

The Airtraq SP NT should not be used, stored or transported at temperatures below 0 °C/32°F or over 40° C/104°F. The relative humidity must not exceed 95 %. The air pressure must not exceed 500 to 1060 hPa.

Airtraq SP NT shelf life is limited to the expiration date.

Airtraq SP NT service life is limited to 40 accumulative minutes, 5 minutes before its end; the Airtraq SP NT will start blinking for 2 seconds every 20 seconds. Once maximum operating time is reached the Airtraq SP NT will blink continuously until battery is depleted.

DISPOSAL

To dispose the Airtraq SP NT once it has been used:

• Separate the eyecup from the main body by pulling it up.

• Remove the battery cover by pulling it away from the main body (pull away from the small notches).

• Remove the batteries from the Airtraq SP NT and place them in an appropriate battery recycling container (dispose of them according to established recycling policies). The batteries are classified as non hazardous waste material and comply with European Directive WEEE.

However, the manufacturer recommends separating them from standard trash.


• Discard the Airtraq SP NT as any other potentially contaminated waste.

MANUFACTURER'S WARRANTY

The manufacturer warrants the Airtraq SP NT against faulty materials or manufacturing defects for only one use or until the expiration date, whichever comes first, provided that the Airtraq SP NT is used in accordance with the procedures set forth in these instructions. This Warranty is applicable only if the device is purchased from an authorized distributor.

The Airtraq SP NT is designed for SINGLE-PATIENT-USE.

This device has not been designed to be cleaned or sterilized. Use beyond this recommendation may generate serious consequences in the product's performance and will void the Airtraq SP's warranty. The manufacturer disclaims all other warranties, whether expressed or implied, including, without limitation, the warranties of merchantability or fitness for a particular use.

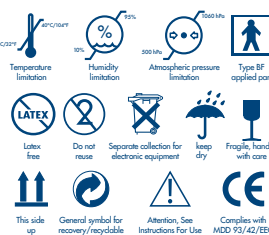Manufactured by:

PRODOL MEDITEC LIMITED

No. 18, 7th Science Ave.,
Hi-Tec Coast, Zhuhai, Guangdong,
519085 P.R. China

USA Representative:

AIRTRAQ LLC

800 Sun Park Dr.
Fenton, MO 63026 USA

EU Representative:
PRODOL MEDITEC, S.A.

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Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.

For further advice on using the Airtraq SP please visit: www.airtraq.com or contact:

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Airtraq is a registered trademark.